


Instructions for Use

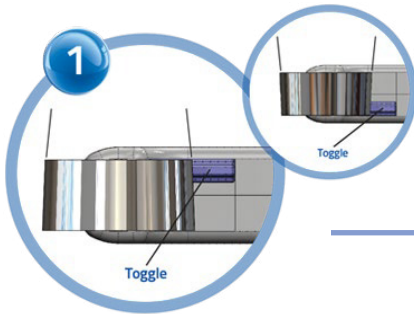

The toggle should be positioned close to the gum line to ensure the correct fit around the tooth. **Before use, adjust the toggle to suit the mouth quadrant in which the Pro-Matrix is being used.**



Fit the band around the tooth and tighten by turning the thumbscrew clockwise. Do not over tighten. For larger teeth, unscrew the band counterclockwise.



Apply wedges and shape the band using a suitable burnishing tool if required.



Perform restoration using normal clinical procedure.



To remove the band, first loosen it by turning the handle 2-3 turns counterclockwise.